

## Audiologists Instructions

- Observe recognized procedures for otoscopic examination of the ear and be certain to screen the client's ear canal for contraindication.
- Insert an oto-block slightly beyond the second directional bend.
- We require the entire outer ear and down to the second bend of the ear canal.
- Please use a Bite Block and require the patient to hold the jaw open about 1 inch while the impression cures.
- Use only silicone impressions material. Impressions will not be processed from powder and liquid type materials.
- Syringe the impressions material into the ear.
- Be sure to fill the canal up to the second directional bend and fill the concha completely.
- Be sure to over-fill the area between the tragus and helix.
- Provide ample time for the impressions to cure completely.
- Impressions must not contain cracks, ripples or voids.
- Carefully pack impressions in a crush resistant box, include a printed copy of your confirmation email and ship them to: 5528 Commerce Dr. Orlando, FL 32839.

